

Instructions for hair sampling DNA analysis


Pull 20-30 hair **with** roots from the switch



Place hair in envelope or plastic bag – hair must be dry




Label the envelope or plastic bag with the ID/name of the animal

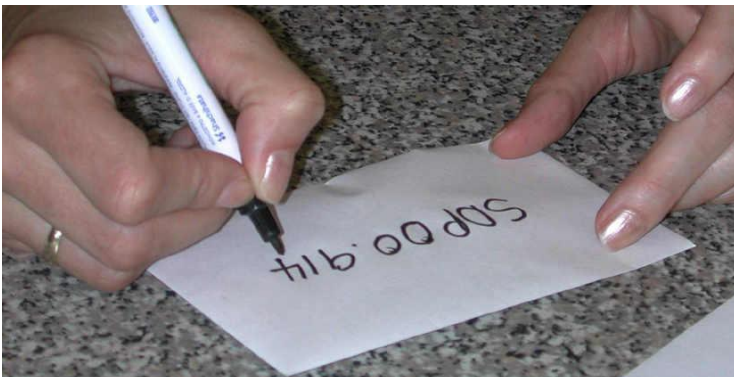


Complete form in full.

For parentages:

Remember complete details of parents

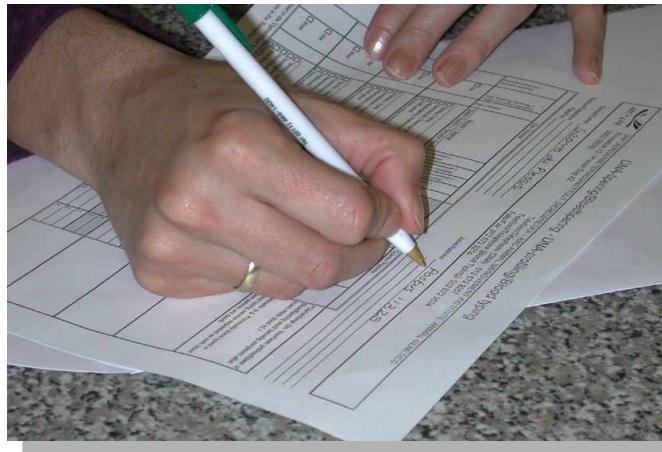

Place hair samples and form in envelope and send by **registered mail or courier to:**

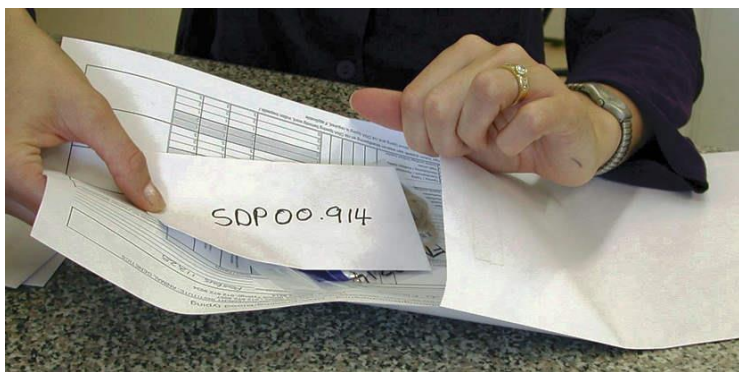

**ARC – Animal Production
Olifantsfontein Road
Animal Genetics – Horses
Att: Pranisha Soma
Private Bag X2
IRENE
0062**

NB: Do not send fine mane/body hair. The quantity and quality DNA obtained is too low for a complete DNA profile and may delay the issuing of results.

